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Instructions

Instructions

Istruzioni

Instrucciones

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© 2018 by peniXtend. Alle Rechte vorbehalten

Instructions

Sincere congratulations for buying the peniXtend product!

peniXtend is a reliable device for enlarging the size of penis through long-term stretching treatment. It is woven in the form of condom technology, and its amazingly easy handling and simple use revolutionize the traditional products in the market.

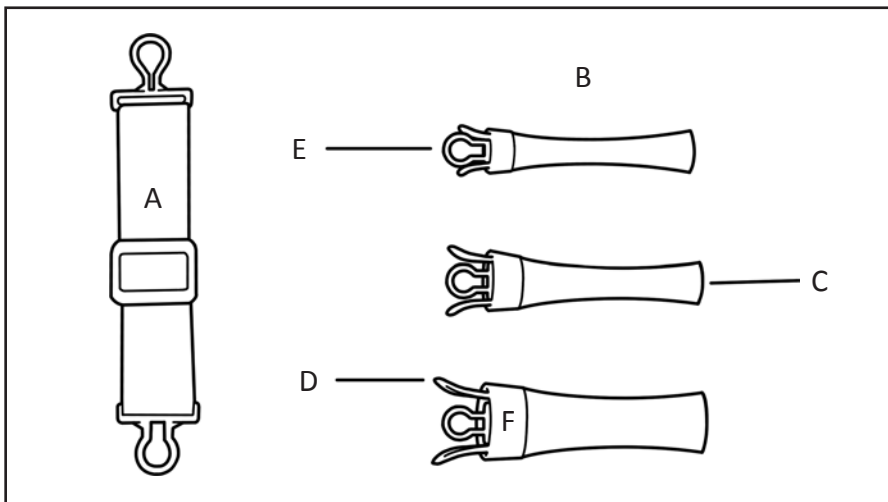
Indications:

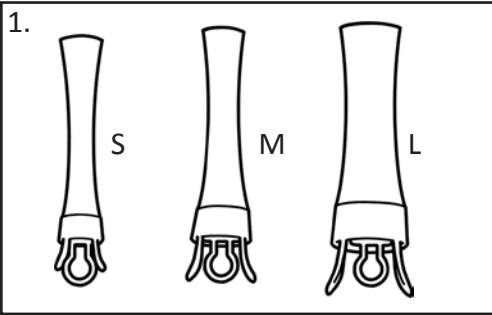
peniXtend can be used in the following cases

- Willing to increase the length or diameter of the penis
- Genetic or non-congenital curvature of the penis
- Circulatory- or impotence-induced erection disorders
- Sexual function practice after prostate surgery
- Desire for energy and more sexual endurance

Package content:


- A 1 x tension belt
- B 3 x tensile strap (condom woven with connector) in different sizes (S/M/L)
- C Funnel-shaped opening
- D Holders, outer part of the woven condom
- E Connector inside the woven condom
- F Pressure ring


Instruction:

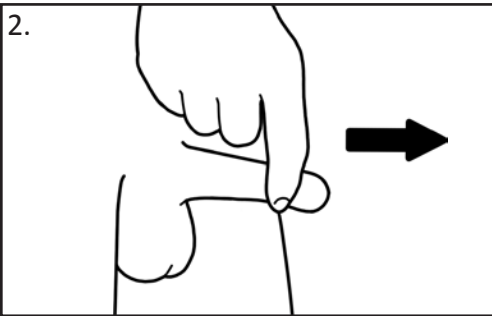
1. Unbox the peniXtend components.
Choose your perfect size (S/M/L).

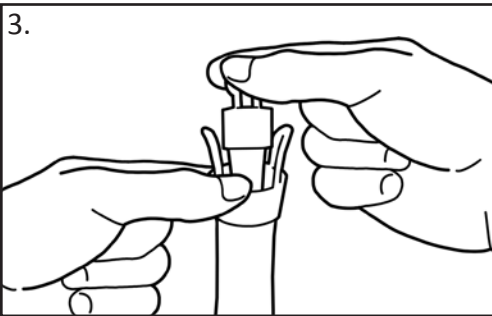2. Hold your glans with your hand and drag it out for about 10 seconds so that the penis is stretched.

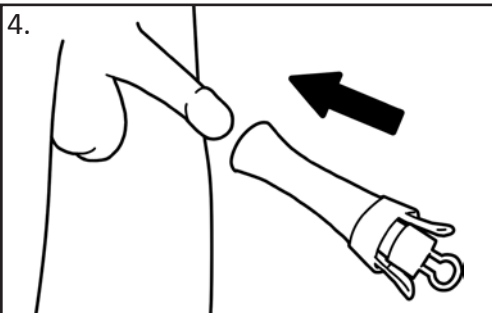
3. Hold the peniXtend by placing one hand on the internal connector and the other on the pressure ring.


4. Place the glans in the funnel-shaped part, (ie) part of the woven condom that is slightly wider, and push the peniXtend gently toward your penis.

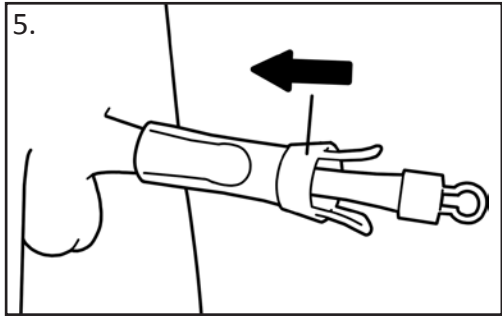
5. Now drag the woven condom over the glans. In this way, the penis is gradually placed completely within the woven condom. The pressure ring, located at the bottom of the woven condom, helps you both during tightening and removing the woven condom.

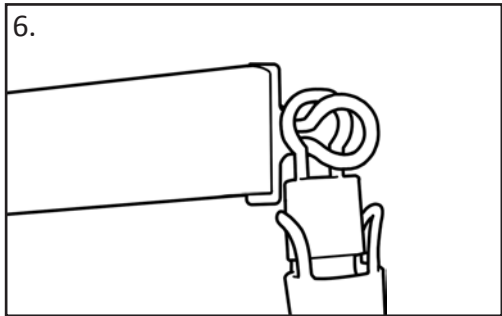
6. To create pressure on peniXtend, use the connector inside the woven condom to attach it to the tension belt.

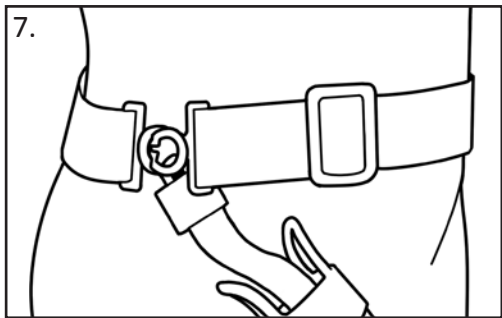7. Then place the tension belt around the lower part of your back and hips, and fix the connector like a belt. To sit comfortably, adjust the belt (loose or stiff) and consider the pressure when adjusting.

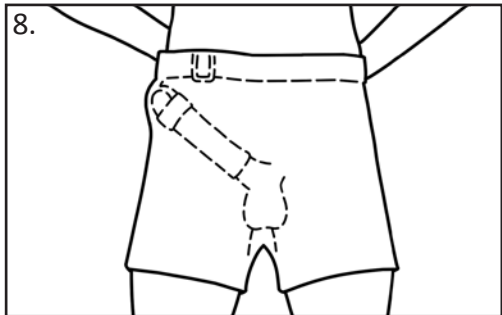8. Wear underclothes and your pants. The peniXtend is now invisible from the outside and can be worn comfortably for up to twelve hours a day.

Precautions and side effects:

Self use of the peniXtend may lead to some problems .The manufacturer cannot be held responsible for damage caused by non-compliance with the safety instructions and warnings, but it can be used safely according to the mentioned instructions. However, the penis should not be over-stretched, especially in the first week of use.

If you experience pain and or swelling on the penis (especially over the glans), you should reduce the tension belt stretching strength. In such cases, it is better to stop using peniXtend until the swelling or pain is reduced. Then the stretching strength should be adjusted to a lower amount. In this way, the success of using this product is not endangered. In fact, the main factor to achieve the desired result is the continuation of the stretching action, rather than the intensity of the stretch.

peniXtend is as a health device designed for adults. Therefore, keep it out of reach of children. To prevent disease and infection, and for health reasons, do not allow third parties to use peniXtend. Do not use peniXtend on affected, contaminated, or stimulated skin. Clean peniXtend regularly with warm water and if necessary with soap and water.

Experience has shown that a regular, long-term stretch with less power leads to a better result than a short-term stretch with intense and even painful strength. Therefore, do not adjust the stretching strength too strong. If you suffer from circulatory problems when applying excessive pressure on your penis, stop using peniXtend for a short time. Then find the best settings for yourself through try and error

As regular tightening is important, rest periods are also important. The desired effect of enlargement does not take place in the stretch period, rather it is due to the resting or growth stage.

Given the overnight erection, it is generally recommended that you do not use peniXtend during sleep, and keep this time for the resting or growth stage.

Both the connector and the woven condom are made of materials which are suitable for patients with allergies and diabetes. Stop using if you have any allergies or irritations. Ask your doctor or pharmacist for caution.

Do not use peniXtend after taking alcohol, drugs, analgesics, sleep pills, or tranquilizers. Because the pain feeling may be negatively affected by them and you may not be able to respond quickly and properly to avoid injury.

On the last two pages of this description, you will find a table that helps you control your penis growth.

Wish you great and entertaining success!

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©2018 peniXtend. All Rights Reserved